Boss Medical Fluid 100

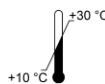
Base-UDI: 42606773310000575663

BG:	Инструкция за употреба	2
CZ:	Návod k použití	5
DA:	Brugsanvisning	8
DE:	Gebrauchsanweisung	11
EL:	Οδηγίες χρήσης	14
EN:	Instructions for Use	17
ES:	Instrucciones de uso	20
FI:	Käyttöohje	23
FR:	Mode d'emploi	26
HR:	Upute za uporabu	29
HU:	Használati útmutató	32
IT:	Istruzioni per l'uso	35
LT:	naudojimo instrukcija	38
NO:	Bruksanvisning	41
PL:	Instrukcja użycia	44
PT:	Instruções de uso	47
RO:	Instrucțiuni de utilizare	50
SK:	Návod na použitie	53
SV:	Bruksanvisning	56

18 ml	– UDI-DI: 4260677330190
50 ml	– UDI-DI: 4260677330152
1 L	– UDI-DI: 4260677330220
5 L	– UDI-DI: 4260677330183

Boss Medical Fluid 100

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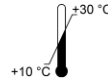
Basis-UDI: 42606773310000575663

Content

1. Application area and intended use of the product.....	17
2. Contraindications and handling to be prevented.....	18
3. Safety instructions.....	18
4. Sterility.....	18
5. Handling and usage	18
6. Storage and shelf life.....	19
7. Composition.....	19
8. Disposal.....	19

Boss Lubricants GmbH & Co. KG
Chemnitzer Straße 15
72458 Albstadt
Γερμανία
Τηλ.: +49 7431 - 70 18 18-0
Φαξ: +49 7431 - 70 18 18-19
Email: vertrieb@boss-oil.de

1. Application area and intended use of the product

Boss Medical Fluid 100 is a high-viscous oil for lubrication and maintenance of medical/surgical instruments. It is applied thinly before sterilization, in particular on stopcocks of valves in endoscopes, and enables a smooth, energy-saving and gentle movement of the instruments. At the same time, *Boss Medical Fluid 100* provides a limited temporary corrosion protection on metal parts.

Boss Medical Fluid 100 is suitable for application on surgical instruments made of metals, PE, PP, POM or PTFE. Information on the compatibility with further materials can be requested from the manufacturer.

After application of the oil the surgical instruments are sterilized by steam (EN ISO 17665-1).

Intended user of this product are manufacturer of medical/surgical instruments and medical staff that is in duty for cleaning, reconditioning and maintenance of medical instruments.

Features of performance:

- Excellent lubrication
- Non-cytotoxic according to EN ISO 10993-5:2009
- Sterilizable by steam according to ISO 17665-1:2006
- High-temperature stable up to 180 °C
- Partly emulsifiable in water
- Free from silicones

Boss Medical Fluid 100

Basis-UDI: 42606773310000575663

2. Contraindications and handling to be prevented

- *Boss Medical Fluid 100* is NOT designed for outer lubrication of endoscopes, specula or the like to support the insertion into body orifices. Also, it is NOT designed for applying it to the skin or the body for to support the wound healing or prevent the skin from drying out (no cream and no caring oil).
- *Boss Medical Fluid 100* is NOT suitable for sterilization by high-energy or ionizing radiation and NOT suitable for sterilization by ethylene oxide.

3. Safety instructions

- Keep away from children.
- Read and follow the instructions for use.
- Note the expiry date.
- Only use prior sterilization.
- Follow the storage conditions.
- Follow recommendations of national institutions for hygiene and infection prevention (e.g. German KRINKO: Hygiene Requirements for the Reprocessing of Medical Devices).

4. Sterility

Boss Medical Fluid 100 is not sterile.

5. Handling and usage

- (1) Hygienically clean and dry the medical instruments according to the manufacturer's specifications.
- (2) Remove the stopcock to be lubricated out of the valve of the endoscope.
- (3) Apply one drop of *Medical Fluid 100* to each of the both opposite sites perpendicular to the through-hole of the stopcock.
- (4) Insert the stopcock into the valve and open and close it two times to spread the oil in the whole valve.
- (5) Remove excess oil with a clean, lint-free cloth. An excessive application of oil can lower the steam penetration of the oil film and lead to a lack of sterilization effect, so that possibly further measures might be necessary for a successful sterilization (longer sterilization duration, validation).
- (6) Optional: pack the medical instrument, if desired.
- (7) Sterilize the medical instrument by steam. Please follow the provisions for the sterilization process according to the manufacturers, harmonized standards and national institutions for hygiene and infection prevention.

Boss Medical Fluid 100

Basis-UDI: 42606773310000575663

6. Storage and shelf life

Protect from direct sunlight and moisture. Store at temperatures from +10 °C to +30 °C (+50 °F to +86 °F) in closed containers.

Note the expiry date printed onto the container (📅). The shelf life is 3 years from production date.

7. Composition

Synthetic white oils, harmless additives.

Boss Medical Fluid 100 is no dangerous good according to Regulation (EU) 1272/2008.

8. Disposal

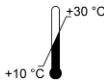
Small amounts of oil may be disposed to residual waste. Bigger amounts of oil should be disposed to an approved waste disposal plant. European waste disposal code: 130206 and 130899.

The oil may be incinerated in waste incineration plants.

Last revision: 22.03.2024

All serious incidents that occurred in connection with the product *Boss Medical Fluid 100*, have to be reported to the manufacturer and the responsible authority of the member state, in which the user and/or patient are resident.

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